
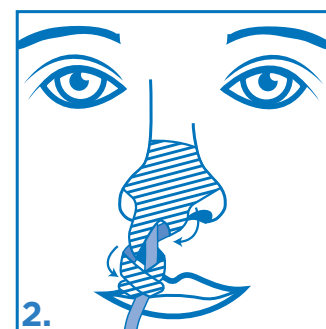
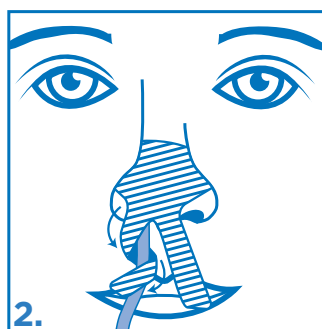
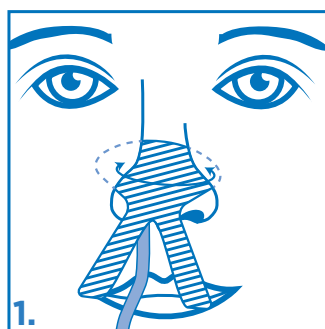


Dale® NasoGastric Tube Holder

Alternate Application Instructions

Application Instructions

1. After the NG tube is in place, please make certain that the skin on the nose and the NG tube is both clean and dry. We recommend cleaning it per hospital protocol.
2. Peel back NG tube holder completely from liner starting at the top of the nose pad. Position and apply nose pad centered on the nose at a slight angle toward the nare that the NG tube has been inserted into. **(see fig. 1)**
3. Choose one of the two tabs closest to the nose and begin to wrap the tube in a spiraling fashion around it so most of the NG tube is centered on the selected nare. **(see fig. 2)**
4. Wrap the second remaining tab using the same spiral technique. **(see fig. 2)** The NG tube needs to hang freely below the nare, not snug up against it.
5. Change the tube holder after 3 days, or sooner if necessary.

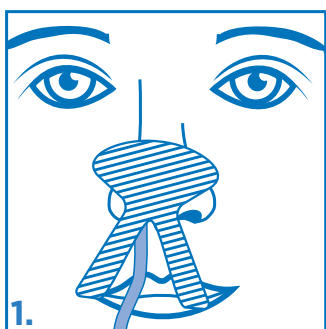

Removal Instructions

1. To remove the NG tube holder from the patient, simply pull back on the non-adherent tips on the end of each tab to unwrap them from the tube.
2. Continue to gently pull the tube holder in an upward fashion to remove from the patient's nose.

* This is a single patient use disposable product

Additional Benefits

- Ideal for holding most nasogastric tubes, including nasoenteric, Salem Sump™, and Levin stomach tubes.
- Skin-friendly adhesive stays in place for up to 3 days, yet is easy to remove.
- Non-adherent plastic tips on both tabs make tube removal safer and easier when donning gloves.

Clinical Studies

The Dale NasoGastric Tube Holder was used in a recent study published in MEDSURG Nursing Magazine. To view the article please visit dalemed.com/ng-study.

ORDERING INFORMATION		
Product Number	Size	Quantity
160	Large/Adult	50/Box
161	Small/Young Adult/Pediatric/ Geriatric	50/Box

Dale[®]
www.dalemed.com

Dale® is a registered trademark of Dale Medical Products, Inc. in the USA & EU.
Salem Sump is a trademark of Covidien AG Corporation in Switzerland.
©2020 Dale Medical Products, Inc.