




Hold-n-Place® Multipurpose Tube Holder

Product Description

A skin friendly, breathable, adhesive backed Holder secures a wide variety of catheters and tubes from 12fr up to 40fr. Provides for either adhesive or adhesive-free securement around tube.

Application Instructions

1. Clean and prepare application area. Apply to clean dry skin.
2. Open Dale Hold-n-Place so that base liners lay flat and center tab is upright. **(A)**
3. Remove base liners to expose adhesive and apply to skin. **(A)**
4. Leave center tab liner in place for adhesive free securement around tube **OR** remove tab liner to expose tube adhesive. **(A)**
5. Place tube at bottom of center tab and securely wrap tab around tube and join tab hook end to loop portion on base. **(B)**
6. To secure Foley catheter, place Y port of the proximal end of the Foley catheter onto center tab and fold over tab to secure. **(C)**

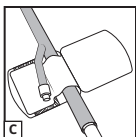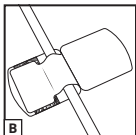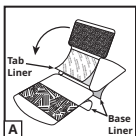

Additional Benefits

- Secures a wide variety of tubes such as: chest tubes, Jackson-Pratt, Foley catheters, drainage tubing, PEG tubes, and multi-port catheters.
- Writeable surface.
- Lasts up to 7 days.

For single patient use. Disposable.

For an application video please visit www.dalemed.com

Dale medical products are available from your preferred hospital/medical supply dealer.

Not made with natural rubber latex.

Patent www.dalemed.com/patents

Dale & Hold-n-Place are registered trademarks of Dale Medical Products, Inc. in the USA, EU, and UK



Dale Medical Products, Inc.
40 Kenwood Circle • Suite 7
Franklin, MA 02038-3298, USA
T: +1-508-695-9316
www.dalemed.com



MDSS GmbH
Schiffgraben 41
D-30175 Hannover, Germany



MDSS CH GmbH
Laurenzenvorstadt 61
5000 Aarau, Switzerland

