




Dale® Hold-n-Place® Multipurpose Tube Holder

APPLICATION INSTRUCTIONS

A skin friendly, breathable, adhesive backed Holder secures a wide variety of catheters and tubes from 12fr up to 40fr. Provides for either adhesive or adhesive-free securement around tube.

APPLICATION INSTRUCTIONS

1. Clean and prepare application area. Apply to clean dry skin.
2. Open Dale Hold-n-Place so that base liners lay flat and center tab is upright. **(A)**
3. Remove base liners to expose adhesive and apply to skin. **(A)**
4. Leave center tab liner in place for adhesive free securement around tube **OR** remove tab liner to expose tube adhesive. **(A)**
5. Place tube at bottom of center tab and securely wrap tab around tube and join tab hook end to loop portion on base. **(B)**
6. To secure Foley catheter, place Y port of the proximal end of the Foley catheter onto center tab and fold over tab to secure. **(C)**

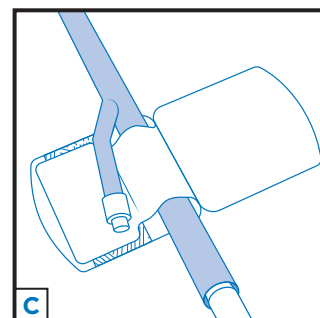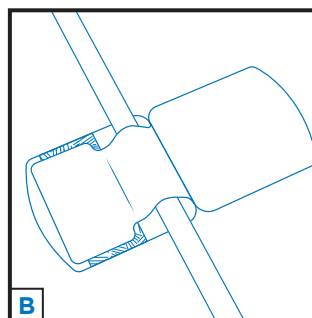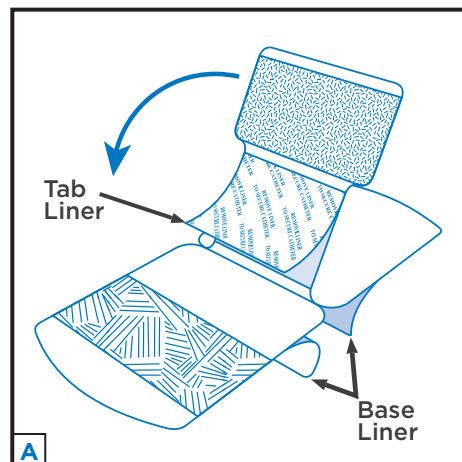

ADDITIONAL BENEFITS

- Secures a wide variety of tubes such as: chest tubes, Jackson-Pratt, Foley catheters, drainage tubing, PEG tubes, and multi-port catheters.
- Writeable surface.
- Lasts up to 7 days.

Product #	Description	Quantity
150	Fits catheters and tubes 12fr-40fr	50/Box

HCPCS A4333



Dale
Medical Products



800-343-3980 • www.dalemed.com

Dale & Hold-n-Place are registered trademarks of Dale Medical Products, Inc. in the USA, EU, and UK.
©2025 Dale Medical Products, Inc. All rights reserved.

SCAN FOR
MORE INFO



AD-039 Rev E