

## NasoGastric Tube Holders

Additional Indications of Use -  
Oral Gastric Tube Holder

### Oral Gastric Tube Holder

1. Be sure the skin is clean and dry.
2. Position device beside patients' mouth with tab(s) towards the mouth.
3. Peel back the liner(s). Apply device to patient.
4. Wrap the tab(s) around the OralGastric Tube.
5. Change holder after 3 days or sooner if necessary.



#### Removal

Gently unspiral from the tube and gently remove from the patient's skin.

**For single patient use. Disposable.**

Dale is a registered trademark of Dale Medical Products, Inc. in the USA, EU, and UK.