


Dale UltraGrip™ NasoGastric Tube Holder

APPLICATION INSTRUCTIONS

APPLICATION INSTRUCTIONS

1. After the tube is in place, make certain that the nose and the NG tube are both clean & dry.
2. Cleanse the nose and area around it with a clean washcloth that has been moistened with warm clean tap water (*no soap*). Towel dry the area immediately after cleaning with a clean dry towel. (*Do not apply any alcohol to this application area*) Alcohol will dry the tissue and may cause micro-cracking of the skin. **(A)**
3. Peel back liner from the **Nose Pad** portion of the holder.
4. Position, align, and apply **Nose Pad** to side of nose where tube exits while keeping tube in center of nare and aligning tube in center of **Foam Bridge Pad** portion. NG Tube Holder should always align with whichever nare is being utilized by the NG Tube. **Single Long Tab** should be parallel to the tube. **(B)**
5. Peel back liner from **Single Long Tab** and begin to wrap **Single Long Tab** in a spiraling fashion so tube is centered within nare. **(C)**
6. Change holder after 3 days or sooner, if necessary.

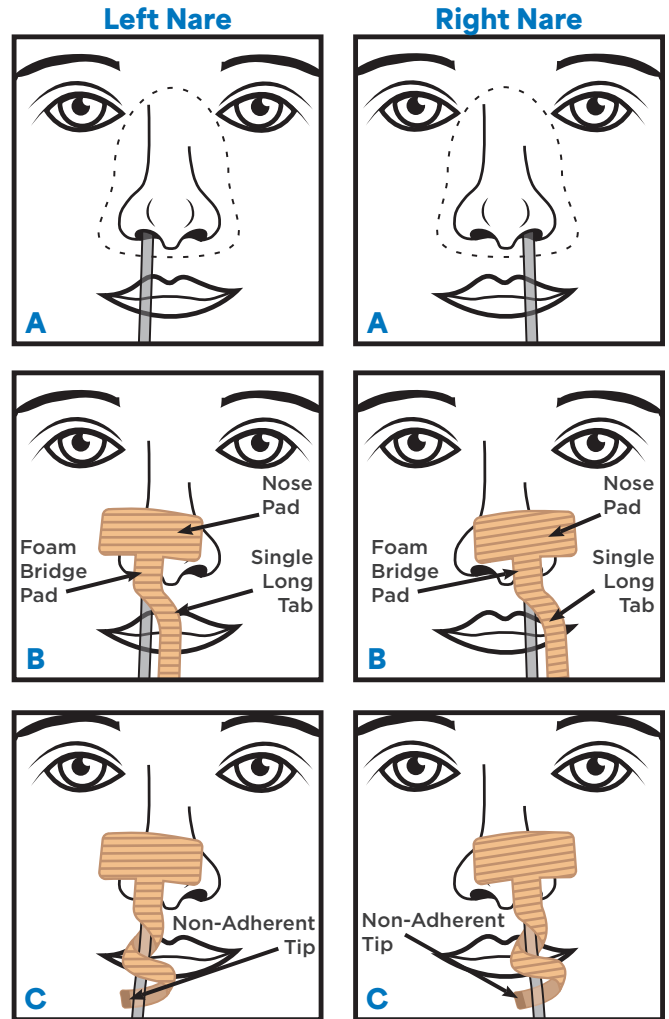

This securement device may be placed to secure NG Tubes in either nare.

REMOVAL INSTRUCTIONS

To remove Holder from the tube, simply pull back on the **Non-Adherent Tip** on the end of the **Single Long Tab**.

Continue to gently unspiral the Holder in an upward fashion to remove from tube and patient's nose.

For single patient use. Disposable.
Not made with natural rubber latex.

Product #	Description	Quantity
163	UltraGrip™ Large/Adult, Single Tab	50/box

Patent dalemed.com/patents



800-343-3980 • www.dalemed.com

Dale is a registered trademark of Dale Medical Products, Inc. in the USA, EU, & UK.
UltraGrip is a trademark of Dale Medical Products, Inc.
©2024 Dale Medical Products, Inc. All rights reserved.

SCAN FOR
MORE INFO



AD-127 Rev A