

UltraGrip™ NasoGastric Tube Holder

Application Instructions

1. After the tube is in place, make certain that the nose and the NG tube are both clean & dry.
2. Cleanse the nose and area around it with a clean washcloth that has been moistened with warm clean tap water (*no soap*). Towel dry the area immediately after cleaning with a clean dry towel. (*Do not apply any alcohol to this application area*) Alcohol will dry the tissue and may cause micro-cracking of the skin. **(A)**
3. Peel back liner from the Nose Pad portion of the holder.
4. Position, align, and apply Nose Pad to side of nose where tube exits while keeping tube in center of nare and aligning tube in center of Foam Bridge Pad portion. NG Tube Holder should always align with whichever nare is being utilized by the NG Tube. Single Long Tab should be parallel to the tube. **(B)**
5. Peel back liner from Single Long Tab and begin to wrap Single Long Tab in a spiraling fashion so tube is centered within nare. **(C)**
6. Change holder after 3 days or sooner, if necessary.

Removal

To remove Holder from the tube, simply pull back on the Non-Adherent Tip on the end of the Single Long Tab.

Continue to gently unspiral the Holder in an upward fashion to remove from tube and patient's nose.

For single patient use. Disposable




For an application video please visit www.dalemed.com

Dale medical products are available from your preferred hospital/medical supply dealer.

Not made with natural rubber latex.

Dale is a registered trademark of Dale Medical Products, Inc. in the USA, EU, and UK.

UltraGrip is a trademark of Dale Medical Products, Inc.

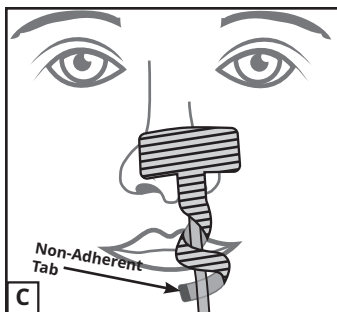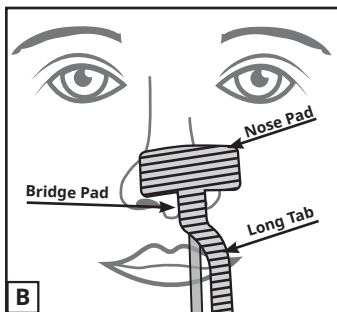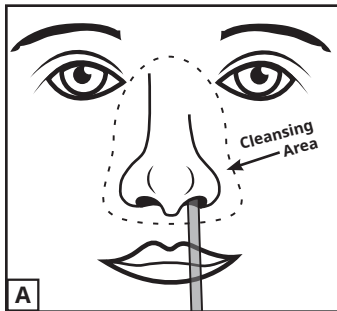

Dale Medical Products, Inc.
40 Kenwood Circle • Suite 7
Franklin, MA 02038-3298, USA
T: +1-508-695-9316
www.dalemed.com



MDSS GmbH
Schiffgraben 41
D-30175 Hannover, Germany



MDSS CH GmbH
Laurenzenvorstadt 61
5000 Aarau, Switzerland

