

Dale® Scrape-n-Suction™ Tongue Cleaner

APPLICATION INSTRUCTIONS


PRODUCT DESCRIPTION

Removes bacteria and toxins in the tongue's biofilm.

APPLICATION INSTRUCTIONS

For best results, use immediately after standard oral care.

1. Attach the barbed end of the Cleaner to the suction tubing.
2. Set suction at **recommended level of 100 mmHg**. Increase suction level as needed to remove thick or encrusted biofilm. **Do not exceed 150 mmHg.**
3. Gently place the head of the Cleaner near the back of the tongue.
4. Gradually, while applying light pressure, draw the Cleaner towards the tip of the tongue, removing the biofilm.
5. Repeat steps 3 & 4 until all areas of the tongue have been cleaned.
6. Discard Cleaner after use.

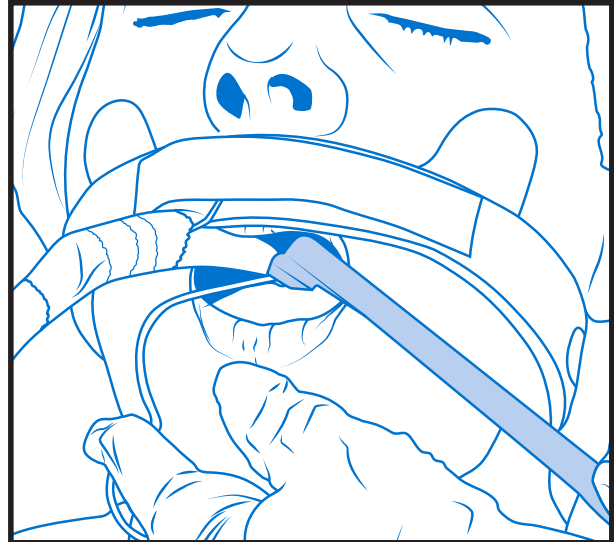

Product #	Description	Quantity
220	Scrape-n-Suction Tongue Cleaner Large/Adult	50/Box

Patent dalemed.com/patents

Dale
Medical Products
800-343-3980 • www.dalemed.com



Dale is a registered trademark of Dale Medical Products, Inc. in the USA, EU, and UK.
Scrape-n-Suction is a trademark of Dale Medical Products, Inc.
©2025 Dale Medical Products, Inc. All rights reserved.

SCAN FOR
MORE INFO



AD-133 Rev A