


## Scrape-n-Suction<sup>™</sup> Tongue Cleaner

### Product Description

Removes bacteria and toxins in the tongue's biofilm.

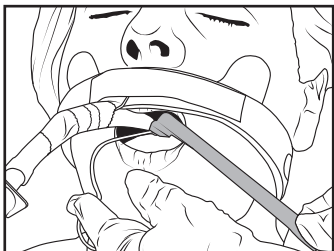

### Application Instructions

For best results, use immediately after standard oral care.

1. Attach the barbed end of the Cleaner to the suction tubing.
2. Set suction at **recommended level of 100 mmHg**. Increase suction level as needed to remove thick or encrusted biofilm. **Do not exceed 150 mmHg.**
3. Gently place the head of the Cleaner near the back of the tongue.
4. Gradually, while applying light pressure, draw the Cleaner towards the tip of the tongue, removing the biofilm.
5. Repeat steps 3 & 4 until all areas of the tongue have been cleaned.
6. Discard Cleaner after use.

For single patient use. Disposable.

For an application video please visit [www.dalemed.com](http://www.dalemed.com)

Dale medical products are available from your preferred hospital/medical supply dealer.

Not made with natural rubber latex.

Patent [www.dalemed.com/patents](http://www.dalemed.com/patents)

Dale is a registered trademark of Dale Medical Products, Inc. in the USA, EU, and UK.

Scrape-n-Suction is a trademark of Dale Medical Products, Inc.



**Dale Medical Products, Inc.**  
40 Kenwood Circle • Suite 7  
Franklin, MA 02038-3298, USA  
T: +1-508-695-9316  
[www.dalemed.com](http://www.dalemed.com)



**MDSS GmbH**  
Schiffgraben 41  
D-30175 Hannover, Germany



**MDSS CH GmbH**  
Laurenzenvorstadt 61  
5000 Aarau, Switzerland

