


Securement of Oxygen Cannula Tubes



The Dale Tracheostomy Tube Holder can be used to secure oxygen cannula tubes




TOP OF HEAD

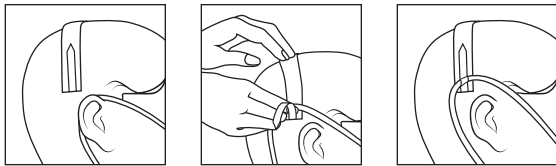

1. With oxygen cannula in place on patient, unfold the holder and place over top of patient's head, and slightly behind ears.
2. Open tabs at end.
3. Place cannula onto device and secure with narrow fastener tabs.
4. Adjust for proper length.
5. Trim excess material (if needed).

BACK OF NECK

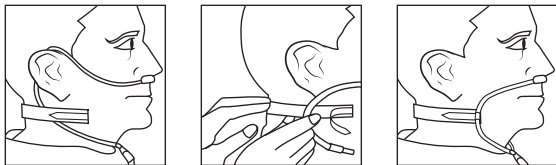

1. With oxygen cannula in place on patient, unfold the holder and center it around back of patient's neck.
2. Open tabs at end.
3. Place cannula onto device and secure with narrow fastener tabs.
4. Adjust for proper length.
5. Trim excess material (if needed).

Benefits

- ◆ Can be worn over the head or behind the neck
- ◆ Helps keep the cannula in place for consistent and uninterrupted oxygen delivery.
- ◆ Relieve pressure and reduce irritation on the ears
- ◆ Adjustable for snug fit
- ◆ Any size can be used to secure the oxygen tubing:
 - + 240 Blue® 1" wide band, fits up to 49.5cm (19.5")
 - + 241 PediPrints® 1" wide band, fits up to 46cm (18")
 - + 242 PediPrints® ¾" wide band, fits up to 23cm (9")
 - + 244 Blue® for bariatric patients 1 ½" wide band, fits up to 71cm (28")