

## Tracheostomy Tube Holders

Additional Indications of Use - Oxygen Cannula

### Securement of Oxygen Cannula Tubes

#### Top of Head



1. With oxygen cannula in place on patient, unfold the holder and place over top of patient's head, and slightly behind ears.
2. Open tabs at end.
3. Place cannula onto device and secure with narrow fastener tabs.
4. Adjust for proper length.
5. Trim excess material (if needed).

#### Back of Neck



1. With oxygen cannula in place on patient, unfold the holder and center it around back of patient's neck.
2. Open tabs at end.
3. Place cannula onto device and secure with narrow fastener tabs.
4. Adjust for proper length.
5. Trim excess material (if needed).

**For single patient use. Disposable.**

Dale is a registered trademark of Dale Medical Products, Inc. in the USA, EU, and UK.