

Dale® Tracheostomy Tube Holders

Application Instructions

1. Thread long, narrow fastener tabs through flanges on trach plate.
2. Bring tabs back over flange and adhere to the neckband.
3. Secure fastener tab of short section (A) to the long neckband section (B). Adjust for proper length or compression.
4. Trim excess material (B only) for proper length.

Caution

Use in supervised care settings only.

Monitor to prevent unintentional disengagement. Do not trim fastener hook tabs.


Recommendations

- Change the holder daily or more frequently as required. Change after patient bathing.

Additional Benefits

- Secure fastener tabs limit movement of the tracheostomy tube reducing accidental decannulation.
- Exclusive moisture wicking neckband lining reduces the risk of skin breakdown by minimizing the amount of moisture absorbed.
- Adjustable two piece neckband.
- PediPrints® series offers kids fun, comforting prints.†
- Fastener tabs quickly and easily attach to any size tracheostomy tube.
- Stretch material‡ allows for cough reflex, accommodates edema, and ensures a snug fit without compromising security.
- Not made with natural rubber latex.

† On the 241 and 242 tube holder
‡ On the 240 Blue® tube holder

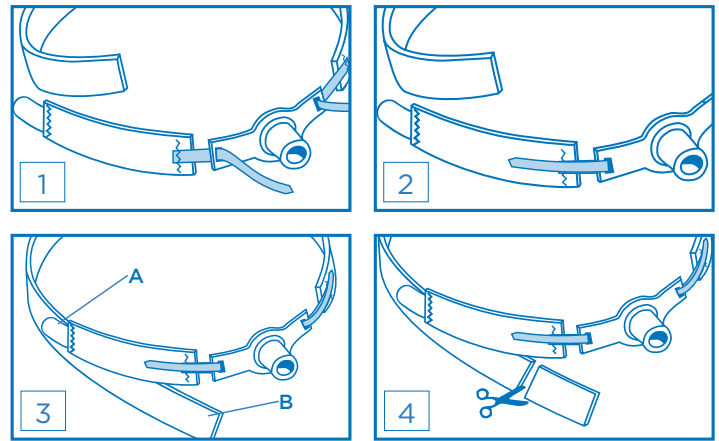

ORDERING INFORMATION

Product Number	Size	Quantity
240	240 Blue® 2.54cm(1") wide, fits up to 49.5cm(19.5") neck	10/Box
241	241 PediPrints® 2.54cm(1") wide, fits up to 45.5cm(18") neck	10/Box
242	242 PediPrints® 1.91cm(0.75") wide, fits up to 23cm(9") neck	10/Box
244	244 Blue® 3.81cm(1.5") wide, fits up to 71cm(28") neck	10/Box

Dale®
www.dalemed.com

Dale®, 240 Blue®, 244 Blue®, & PediPrints® are registered trademarks of Dale Medical Products, Inc. in the USA & EU.
©2020 Dale Medical Products, Inc.

AD-097 REV A