


Tracheostomy Tube Holders

Application Instructions

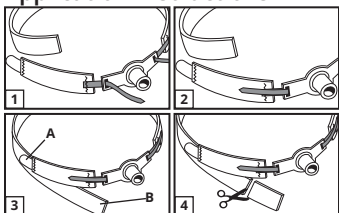

1. Thread long, narrow fastener tabs through flanges on trach plate.
2. Bring tabs back over flange and adhere to the neckband.
3. Secure fastener tab of short section **(A)** to the long neckband section **(B)**. Adjust for proper length or compression.
4. Trim excess material **(B only)** for proper length.

Caution

Use in supervised care settings only. Monitor to prevent unintentional disengagement. Do not trim fastener hook tabs.

Change the holder daily or more frequently as required. Change after patient bathing.

For single patient use. Disposable

For an application video please visit www.dalemed.com

Dale medical products are available from your preferred hospital/medical supply dealer.

Not made with natural rubber latex.

Patent www.dalemed.com/patents

Dale is a registered trademark of Dale Medical Products, Inc. in the USA, EU, and UK



Dale Medical Products, Inc.
40 Kenwood Circle • Suite 7
Franklin, MA 02038-3298, USA
T: +1-508-695-9316
www.dalemed.com



MDSS GmbH
Schiffgraben 41
D-30175 Hannover, Germany



MDSS CH GmbH
Laurenzenvorstadt 61
5000 Aarau, Switzerland

