

Dale Abdominal Binder[®]

with EasyGrip[™] Strip

APPLICATION INSTRUCTIONS

APPLICATION INSTRUCTIONS






Dale Abdominal Binders are available in a range of sizes. Two binders can be joined to fit large patients.

1. Place the binder under the patient with hook material facing up and V point facing down and stretch one end out. Proper stretch provides necessary compression.
2. Stretch the binder over the patient's stomach and hold with one hand. The binder may be placed straight, or at an angle if additional pressure is needed on the upper or lower abdomen.
3. Stretch the other end over at the desired angle.
4. Press the hook firmly closed while stretching down to close the material.


Be sure to stretch the binder adequately. Remember, a secure fit won't restrict breathing; the stretching and compression actually facilitates deep breathing.

USING DRAINAGE TUBES

A hole can be cut in the Dale Abdominal Binder to accommodate tubes such as drainage and feeding. Place the binder on the patient so that there is only a single thickness over the drainage site. Mark an X on the fabric where the hole should be made, crease the fabric and make an elliptical cut in the material (see illustration). When the fabric is stretched, the elliptical cut will become a round hole to accommodate the drain. The diameter of the hole will be equal to the length of the cut.

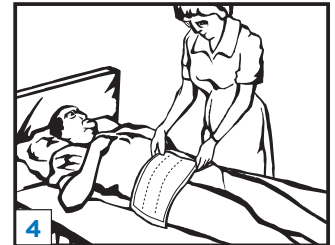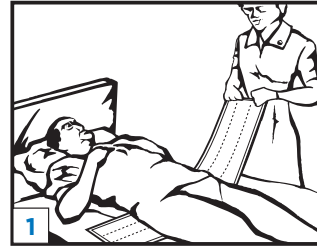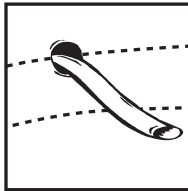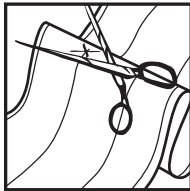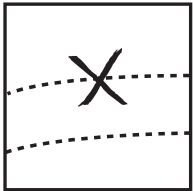

APPLYING THE EASYGRIP STRIP

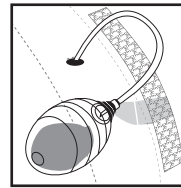

1. Lift EasyGrip Strip off Binder
2. For drainage bulb use, thread EasyGrip Strip through drainage bulb tab; reattach to binder.

†Always follow best practice standards for drainage devices.



Product #	Description	Quantity
525	6" Wide, Stretches to fit 72cm-127cm (28"-50") *can also be used as a rib belt	1 each
410	9" Wide, Stretches to fit 76cm-114cm (30"-45")	1 each
411	9" Wide, Stretches to fit 117cm-157cm (46"-62")	1 each
418	9" Wide, Stretches to fit 152cm-191cm (60"-75")	1 each
810	12" Wide, Stretches to fit 76cm-114cm (30"-45")	1 each
811	12" Wide, Stretches to fit 117cm-157cm (46"-62")	1 each
818	12" Wide, Stretches to fit 152cm-191cm (60"-75")	1 each
820	12" Wide, Stretches to fit 183cm-239cm (72"-94")	1 each
920	15" Wide, Stretches to fit 183cm-239cm (72"-94")	1 each

HCPCS Codes A4463, L0625

Dale
Medical Products



800-343-3980 • www.dalemed.com

Dale is a registered trademark of Dale Medical Products, Inc. in the USA, EU, and UK.
EasyGrip is a trademark of Dale Medical Products, Inc.
©2024 Dale Medical Products, Inc. All rights reserved.

SCAN FOR
MORE INFO



AD-070 Rev B