

# DALE<sup>®</sup> ACE CONNECTOR<sup>®</sup> WITH ENFit<sup>®</sup> CONNECTION SYSTEM

infuse nutrients, medications, fluids, irrigate, check gastric residuals and control flow.

**PRODUCT NUMBER**

**485**

## **AUTO-SEAL SYRINGE MED PORT**

accepts most 60mL ENFit type tip syringes for irrigation, medication delivery and checking gastric residuals.\*\*

Align, Push & Twist  
ENFit 60mL syringe  
or Administration  
Set ENFit female  
connection into ENFit  
Auto-Seal Syringe  
Med Port with male  
ENFit Connection.

\*\*Medication must be crushed properly to prevent potential clogging.

## **DISTAL END (PATIENT END)**

closest to patient's stomach.

- universally connects to NG small bore tubes with ENFit connectors.

## **ENFit FEMALE CONNECTOR CONNECTS TO:**


- Small bore feeding tubes with male ENFit connectors.
- PEG, PEJ, and most G-J tubes with male ENFit connectors.
- Balloon replacement tubes with male ENFit connectors.

## **ON/OFF HANDLE**

controls fluid flow.

"ON" for feeding.

"OFF" to irrigate tube, check gastric residuals, administer medication, or ambulate patient.

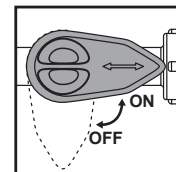

## **PROXIMAL END (FEEDING BAG END)**

furthest from patient.

## **ENFit MALE CONNECTOR CONNECTS TO:**

- enteral pump and gravity delivery sets with ENFit female connection.
- bolus feeding syringes with ENFit tip.

## **THE ACE ADVANTAGE**

- Supports a closed enteral system
- Protects clinician from potentially infectious splash-back
- Protects patient from outside contamination
- No soiled bed linens or loss of formula due to open adapters
- Provides auto-seal syringe med port seal for accepting medication, irrigation, or gastric residuals

**Scan For  
More Info**



**MR CONDITIONAL**

Note: the distal end is always on the side closest to the pt. This is true from a clinician's perspective. It is exactly opposite from a patient's point of view.

# Dale<sup>®</sup>

**800-343-3980**

**www.dalemed.com**

Dale<sup>®</sup> and ACE Connector<sup>®</sup> are registered trademarks of Dale Medical Products, Inc. in the USA & EU.  
ENFit<sup>®</sup> is a registered trademark of GEDSA used with their permission.  
©2018 Dale Medical Products, Inc.