


Dale ACE Connector® with ENFit®

FEEDING PROCEDURE APPLICATIONS

DISTAL END (PATIENT END)

Closest to patient.

- Universally connects to all ENFit ported NG small bore Dobbhoff, Medium Levin tubes, large bore Salem Sumps, & all Stomach tubes.

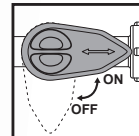

PRODUCT #
485

PROXIMAL END (FEEDING BAG END)

Furthest from patient.

- Universally connects to all ENFit feeding sets with ENFit ports.

ENFit Dobbhoff



- ENFit ported Dobbhoff Feed tube
- Dale ACE ENFit 485
- ENFit ported Feeding Admin Set

Legacy Levin Tube



- Legacy Levin tube for feeding
- ENFit Stepped Transition Connector
- Dale ACE ENFit 485
- ENFit ported Feeding Admin Set

ENFit PEG, PEJ, BRT Tube



- ENFit ported PEG, PEJ, or BRT tube
- Dale ACE ENFit 485
- ENFit ported Feeding Admin Set

ENFit BD Salem Sump™



- ENFit BD Salem Sump with integrated ENFit ports
- Dale ACE ENFit 485
- ENFit ported Feeding Admin Set

ENFit Cardinal Salem Sump™



- Cardinal Salem Sump with tethered ENFit transition adapter
- ENFit transition adapter must be inserted into Salem
- Dale ACE ENFit 485
- ENFit ported Feeding Admin Set



Dale
Medical Products

800-343-3980 • www.dalemed.com

Dale and ACE Connector are registered trademarks of Dale Medical Products, Inc.
ENFit is a registered trademark of GEDSA used with their permission.
Salem Sump is a trademark of Cardinal Health, Inc.
©2023 Dale Medical Products, Inc. All rights reserved.



AD-119 Rev A

*Note: The distal end is always on the side closest to the pt. This is true from a clinician's perspective. It is exactly opposite from a patient's point of view.