

(Works Controller for Enteral)

\*Access with open or closed enteral systems that are ISO 80369-3 compliant

**Directions for Use**

**1 Feeding Procedure** can be used with ENFit type small bore feeding tubes including *Dobhoff*, PEG, PEJ, BRT Gastrostomy, G/J, feeding administration sets, and *Special Salem Sump™* tubes equipped with ENFit Connectors. When ACE Connector is used for enteral feeding, attach female ENFit distal port of ACE to male ENFit Connector of an enteral tube. Follow instructions a-d below.

- Attach female ENFit port (distal side) of male ENFit connector on patient feeding tube.
- Align & twist enteral feeding set with ENFit female connector onto male ENFit port (proximal side) of **ACE Connector**.
- Turn handle to the ON position to deliver feeding formula.
- Turn handle to the OFF position when feeding is complete and remove feeding set from the **ACE Connector** by untwisting.

\*Balloon-less PEG tubes may require the use of a white 80369-3 ENFit transition connector or a universal adapter, not supplied with the **ACE Connector**.

**2 60mL ENFit tip irrigation syringe/Lavage and liquid medication delivery procedure with full threads.**


As a safety feature, the new **ACE Connector** will only accept ENFit tip enteral irrigation syringes and will not accept catheter tip syringes, Luer tip syringes, Luer tip administration set, or hypodermic needles.

- Turn handle to OFF position.
- While holding the **ACE Connector** firmly Align, Push & Twist the ENFit female tip syringe into the Auto-Seal Syringe Med Port until it stops. - **DO NOT OVERTIGHTEN!** It is highly recommended to flush before and after use.
- Dispense fluid or medication into the **ACE Connector** auto-seal syringe med port or withdraw gastric residuals if preferred.
- Remove syringe. The Auto-Seal Syringe Med Port with male ENFit Connector interface will seal automatically.
- Flush with water through Auto Seal Syringe Med Port until clear of any medication residue.
- Turn handle to ON.

This product is compatible with ENFit stepped transition connector and ENFit funnel connectors.

**3 Drainage & Suction Procedure**

The ACE 485 Enteral Connector can be used for suction and drainage with specially equipped ENFit Salem Sump tubes and Suction tubes with a special ENFit adapter.

**4 MR Conditional**  Non-clinical testing demonstrated that the Dale Medical 485 ENFit ACE CONNECTOR is MR Conditional. A patient with this device can be scanned safely in an MR system immediately after placement under the following conditions:

- Static magnetic field of 1.5-Tesla and 3-Tesla, only.
- Maximum spatial gradient magnetic field of 3000 Gauss/cm or less.

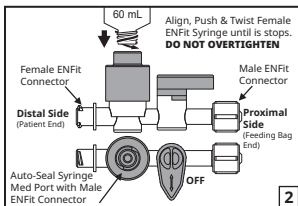
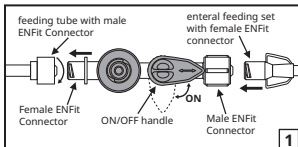
For single patient use / Store in a cool dry place

Federal (USA) law restricts this device to sale by or on the order of a physician.

- To ambulate patient, turn handle to OFF position and disconnect feeding set. Patient can then ambulate as desired. Feeding can resume as needed by repeating above procedures.
- Change as needed as per hospital protocol. Not recommended for use over 30 days.
- In the event that the device is not performing per instruction or as expected, thoroughly flush the **ACE Connector** per instructions and check enteral delivery setup. If the problem persists, replace with a new **ACE Connector** and enteral delivery set as required, following hospital "enteral feeding management" protocol.
- ACE Connector** primarily provides a delivery method for patient enteral feeding. When delivering medications follow hospital or facilities enteral feeding management protocol.

Dale medical products are available from your preferred hospital/medical supply dealer.

Patent [www.dalemed.com/patents](http://www.dalemed.com/patents)

**Ordering Information****485**

Dale ACE Connector  
Individually packaged.  
Sold in boxes of 50.



**Dale Medical Products, Inc.**  
40 Kenwood Circle • Suite 7  
Franklin, MA 02038-3298, USA  
T: +1-508-695-9316  
[www.dalemed.com](http://www.dalemed.com)



Not made with  
natural rubber latex

**Dale**  
Medical Products  
942253-ART\_REV\_G

Made in China

Dale & ACE Connector are registered trademarks of Dale Medical Products, Inc. in the USA, EU, and UK  
ENFit is a registered trademark of GEDSA used with their permission.  
Salem Sump is a trademark of Cardinal Health, Inc.  
©2024 Dale Medical Products, Inc.

English