


Dale ACE Connector® with ENFit®

SUCTION PROCEDURE APPLICATIONS

DISTAL END (PATIENT END)

Closest to patient.

- Universally connects to NG large bore tubes with ENFit connectors or adapters.
- PEG, PEJ, BRTs, and most G-J tubes with male ENFit connectors.

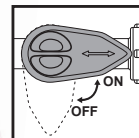

PRODUCT #
485

PROXIMAL END (FEEDING BAG END)

Furthest from patient.


- Connects to standard suction tubes with BD ENFit adapter.

ENFit BD Salem Sump™



- ENFit BD Salem with integrated ENFit ports
- Dale ACE ENFit 485
- BD ENFit Suction Adapter #EN0066000
- Std Suction Tubing.

ENFit Cardinal Salem Sump™

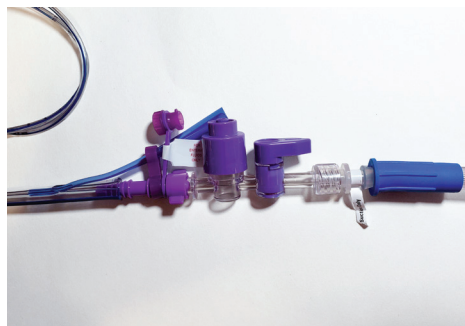


- Cardinal Salem with tethered ENFit Transition Stepped Adapter
- Dale ACE ENFit 485
- BD ENFit Suction Adapter #EN0066000
- Std Suction Tubing.

ENFit PEG, PEJ, BRT



- ENFit PEG/PEJ Tube
- Dale ACE ENFit 485
- BD ENFit Suction Adapter #EN0066000
- Std Suction Tubing.

Legacy Levin Tube

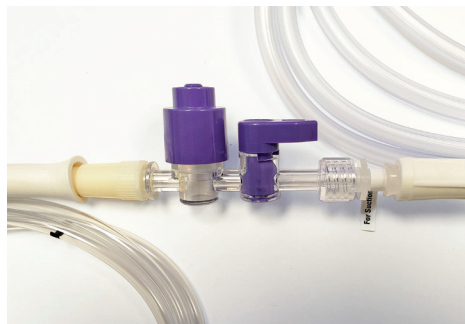

- Legacy Levin Tube with ENFit Transition Stepped Adapter
- Dale ACE ENFit 485
- BD ENFit Suction Adapter #EN0066000
- Std Suction Tubing.



Dale
Medical Products

800-343-3980 • www.dalemed.com

Dale and ACE Connector are registered trademarks of Dale Medical Products, Inc.
ENFit is a registered trademark of GEDSA used with their permission.
Salem Sump is a trademark of Cardinal Health, Inc.
©2023 Dale Medical Products, Inc. All rights reserved.



SCAN FOR
MORE INFO



*Note: The distal end is always on the side closest to the pt. This is true from a clinician's perspective. It is exactly opposite from a patient's point of view.

AD-120 Rev A