


IV Line Manager


Intended to separate and organize multiple IV lines (up to 6) connected to a single patient.

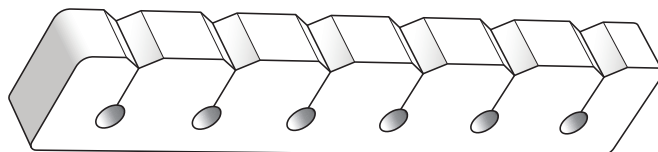

Usage Instructions


Single Device Set-Up

1. Insert each IV tube into its designated slot one at a time.
2. Check all tubing for kinks or twists.
3. You may place the device:
 - ▷ Hanging on the IV tubing.
 - ▷ Lying flat on the patient's bed.

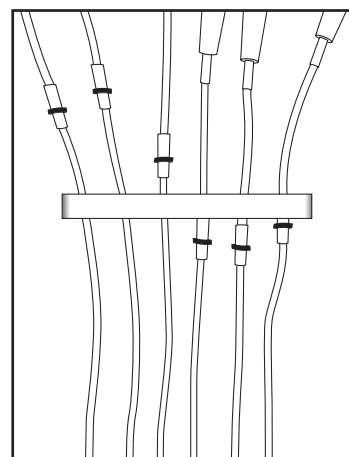

Multiple Device Set-Up

1. Use up to three IV Line Manager devices at the following locations:
 - ▷ Below the infusion pump. (Proximal)
 - ▷ Midway along the IV tubing. (Intermediate)
 - ▷ Next to the lowest injection port near the patient. (Distal)
2. Insert each tube into its corresponding slot on each device.
3. Check all tubing for kinks or twists after insertion.

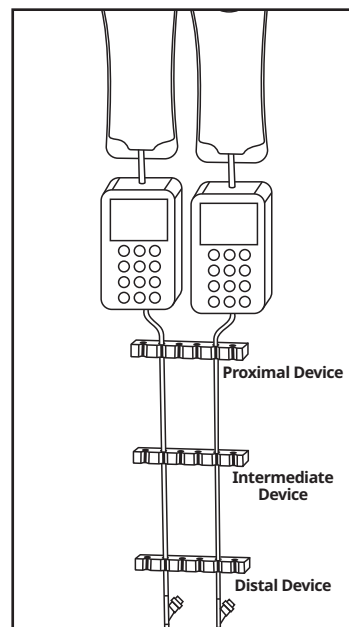

For single patient use. Disposable.

Notice to the user and/or patient; any serious incident that has occurred in relation to this device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Patent www.dalemed.com/patents

Dale is a registered trademark of Dale Medical Products, Inc. in the USA, EU, and UK.



Dale Medical Products, Inc.
40 Kenwood Circle • Suite 7
Franklin, MA 02038-3298
T: 508-695-9316



MDSS GmbH
Schiffgraben 41
D-30175 Hannover, Germany



MDSS CH GmbH
Laurenzenvorstadt 61
5000 Aarau, Switzerland