



Bendable ArmBoard

Application Instructions

1. Align wrist with ArmBoard and bend to desired position.
2. Wrap straps around hand and board to desired tension. Press hook fasteners to secure.

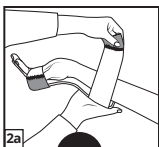

or

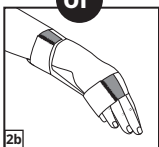

MR Conditional

The Dale Bendable ArmBoard, #650 Large was determined to be MR-conditional.

The findings of the MRI results for the dale 650 ArmBoard apply to the smaller versions (#651, #652, and #653) of the ArmBoards, made from the same materials.

Non-clinical testing demonstrated that the Dale Bendable ArmBoard, #650 Large is MR Conditional. A patient with this device can be scanned safely immediately after placement under specific conditions.

For single patient use. Disposable

For an application video please visit www.dalemed.com

Dale medical products are available from your preferred hospital/medical supply dealer.

Not made with natural rubber latex.

Dale is a registered trademark of Dale Medical Products, Inc. in the USA, EU, and UK



Dale Medical Products, Inc.
40 Kenwood Circle • Suite 7
Franklin, MA 02038-3298, USA
T: +1-508-695-9316
www.dalemed.com



MDSS GmbH
Schiffgraben 41
D-30175 Hannover, Germany



MDSS CH GmbH
Laurenzenvorstadt 61
5000 Aarau, Switzerland

