

Postoperativni Grudnjak

Upute za Primjenu

1. Odvojite naramenice i postavite grudnjak pod bolesnicu.
2. Obuhvatite obje strane grudnjaka i pritisnite ih zajedno. Trebalo bi biti malo napetosti.
3. Naramenice pričvrstite na vrhovima prema prednjem dijelu grudnjaka.
4. Grudić se lako podešava odvajanjem čičak-traka sprijeda i podvezica.



Alternativna Primjena

Dale postoperativni grudnjak može se također postaviti bez odvajanja naramenica, nježno provodeći ruke pacijenta kroz naramenice.

Pranje i održavanje



Prati u toploj vodi.



Sušiti na zraku.



Ne izbjeljivati.



Ne peglati.



Ne čistiti kemijski.

Za uporabu na samo jednom bolesniku. Jednokratno.

Video s uputama za primjenu nalazi se na www.dalemed.com

Ne sadrži prirodni gumeni lateks.

Napomena korisniku i/ili bolesniku: svaki ozbiljan incident koji se dogodi povezano s ovom napravom mora se prijaviti proizvođaču i nadležnom tijelu države članice u kojoj je korisnik i/ili bolesnik registriran ili rezident.

Dale je registrirani žig tvrtke Dale Medical Products, Inc. u SAD-u, EU-u i UK-u



Dale Medical Products, Inc.

40 Kenwood Circle • Suite 7
Franklin, MA 02038-3298, USA
T: +1-508-695-9316
www.dalemed.com



MDSS GmbH

Schiffgraben 41
D-30175 Hannover, Germany



MDSS CH GmbH

Laurenzenvorstadt 61
5000 Aarau, Switzerland

