

Post-Surgical Bra

Application Instructions

1. Detach straps and place the bra under the patient.
2. Grasp both sides of the bra and press together. There should be some tension.
3. Attach straps to front of bra at the top of the band.
4. Bra can easily be adjusted by peeling fasteners apart at front and straps.



Alternative Application

The Dale Post Surgical Bra may also be applied without detaching the straps by gently guiding the patient's arms through the straps.

Laundering & Care



Machine wash warm.



Line Dry.



Do not bleach.



Do not iron.



Do not dry clean.

For single patient use. Disposable

For an application video please visit www.dalemed.com

Not made with natural rubber latex.

Dale is a registered trademark of Dale Medical Products, Inc. in the USA, EU, and UK



Dale Medical Products, Inc.

40 Kenwood Circle • Suite 7
Franklin, MA 02038-3298, USA
T: +1-508-695-9316
www.dalemed.com



MDSS GmbH

Schiffgraben 41
D-30175 Hannover, Germany



MDSS CH GmbH

Laurenzenvorstadt 61
5000 Aarau, Switzerland

