
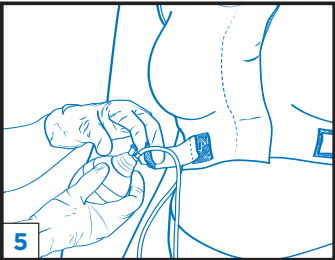
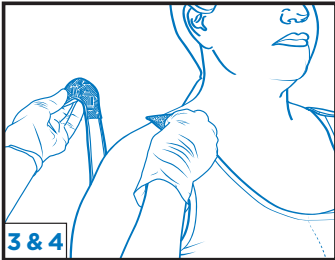
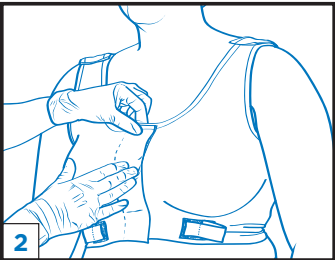


# Dale® Post-Surgical Compression Bra

## APPLICATION INSTRUCTIONS

### APPLICATION INSTRUCTIONS

1. Detach straps and place the bra under the patient.
2. Position the bra so that the hook and loop fasteners are centered over the chest. Press hook and loop fasteners firmly to ensure a secure, comfortable fit.
3. Attach straps to front of bra at the top of the band.
4. For further adjustments, gently open and refasten the hook and loop closure on the shoulders.
5. To secure drainage bulbs, open the tab, slide the tab through the sleeve of the bulb and press to secure.

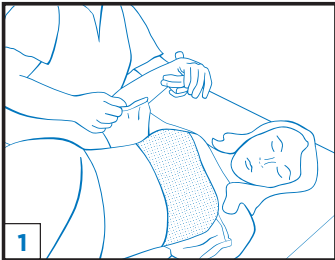


### ALTERNATIVE APPLICATION

The Dale Post-Surgical Compression Bra may also be applied without detaching the straps by gently guiding the patient's arms through the straps.

### PRODUCT FEATURES

- Soft, seamless design helps prevent trauma.
- Multi-directional stretch provides uniform coverage over patient's chest and back.
- Tabs to secure drainage bulbs directly to the bra.
- Material expands and contracts to accommodate edema.
- Hook & loop front closure permits easy application and inspection.
- Provides maximum comfort throughout recovery, even while sleeping.
- Not made with natural rubber latex.

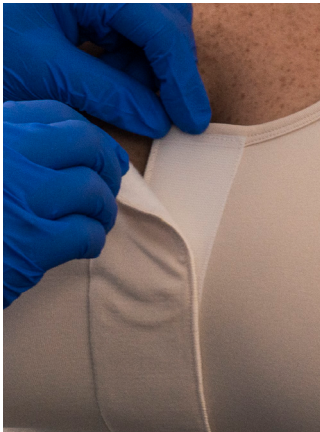

| Product # | Description                                       | Quantity  |
|-----------|---------------------------------------------------|-----------|
| 712       | <b>Medium</b> , fits B-D 86cm-92cm (34"-36")      | 1/Package |
| 713       | <b>Large</b> , fits B-D 92cm-97cm (36"-38")       | 1/Package |
| 714       | <b>X-Large</b> , fits B-D 97cm-112cm (38"-44")    | 1/Package |
| 715       | <b>XX-Large</b> , fits C-E 117cm-137cm (46"-54")  | 1/Package |
| 716       | <b>XXX-Large</b> , fits E-I 137cm-152cm (54"-60") | 1/Package |

HCPCS Codes A4467



800-343-3980 • [www.dalemed.com](http://www.dalemed.com)  
Dale is a registered trademark of Dale Medical Products, Inc. in the USA, EU, and UK.  
©2026 Dale Medical Products, Inc. All rights reserved.



AD-138 Rev A