

Post-Surgical Compression Bra

Application Instructions

1. Detach straps and place the bra under the patient.
2. Position the bra so that the hook and loop fasteners are centered over the chest. Press hook and loop fasteners firmly to ensure a secure, comfortable fit.
3. Attach straps to front of bra at the top of the band.
4. For further adjustments, gently open and refasten the hook and loop closure on the shoulders.
5. To secure drainage bulbs, open the tab, slide the tab through the sleeve of the bulb and press to secure.

Alternative Application

The Dale Post-Surgical Compression Bra may also be applied without detaching the straps by gently guiding the patient's arms through the straps.

Laundering & Care



Machine wash warm.



Line Dry.



Do not bleach.



Do not iron.



Do not dry clean.

For single patient use. Disposable

For an application video please visit www.dalemed.com

Not made with natural rubber latex.

Notice to the user and/or patient; any serious incident that has occurred in relation to this device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Dale is a registered trademark of Dale Medical Products, Inc. in the USA, EU, and UK

